

# ELECTRONIC BABY SCALE

MODEL: SHAEBSA-20

## Operation Manual

### 1. Technical specifications

- Work temperature: 10 ~ 40°C
- Environmental humidity: <80%
- Max. capacity: 20kg
- Graduation: 5g
- Power supply: 4 sections of high quality dry cell
- Low power: L o o
- Error of indication: Err
- Unit change: kg or lb
- Memory function: optionally save and recall weighing value
- Turn off automatically

### 2. Keys' functions

- **ON/ZERO/OFF**: Press the key to power on the scale and start up weighing process. For zero operation, press the key again to clear the display and "zero" enunciator will show in the left low corner of LCD. If you want to turn off the scale please hold pressing this key for about 3 seconds.
- **MODE**: Weighing units exchange key. Press the key to select kg and lb in turn. The suitable indicator of unit is shown in LCD right side by kg or lb enunciator.
- **MEM**: While the scale is in "0.000" situation, you can press this key to recall the former one weighing value saved in the memory and the suitable indicator is lighted. During weighing if the key is pressed and the stable value will be saved, otherwise twinkling value is not saved in the memory.

### 3. Operation

- After the batteries placed, press "**ON/ZERO/OFF**" key to start the scale until "**0.000**" appearance.
- Gently place baby on the pan, weighing record will be flashed on the window after a few seconds. If baby remain lying on the pan after its weight value has shown, the weighing value will continue to flash for 5 minutes and then turn off automatically.
- If take baby from the pan immediately after baby's weight has been flashed, the scale will return to "**0.00**" and then turn off automatically within 5 minutes if no operation again.
- If see about the former saved weighing value, please press the "MEM" key while the scale is in "0.00" position. The scale will save optional value during each weighing operation.

### 4. Cautions

- The scale should be placed on a hard flat surface for baby's easement and safety
- Please replace batteries immediately when window display "L o o"
- Keep the scale clean to ensure the normal working.
- Take the batteries out if no operation for a long term.
- Please place baby in the center of the pan otherwise probably affect weighing accuracy.
- Overload (heavier than 20.5kg) is strictly prohibited.